



# IQ CARE WOUND SOLUTION

## FEATURES

**IQ CARE WOUND SOLUTION** is non-cytotoxic and aids in the healing process of wounds by ensuring fibroblast viability, tissue granulation and epithelialization with minimal scar formation.

This isotonic solution contains pure HOCl ( $\pm 0.02\%$ ) – A powerful yet friendly broad-spectrum hypoallergenic cleansing agent.

Antimicrobial - Kills 99.99% of bacteria, bacterial spores, viruses and fungi on contact, ensuring microbial load reduction in acute and chronic wounds.

No resistance to HOCl has ever been reported and HOCl solutions have been shown to assist in the treatment of:

- Diabetic foot ulcers
- Decubital ulcers
- Venous ulcers
- Burn wounds
- Surgical wounds, etc.

Removes exudate, slime or biofilm by dislodgement due to the **ORP** (Oxidation Reduction Potential).

Reduction and even elimination of costly topical antibiotic substances with known cytotoxic properties.

<b>METHOD OF APPLICATION</b>	In order to achieve a rapid reduction in microbial load, rinse the wound with copious amounts of <b>IQ CARE WOUND SOLUTION</b> until loose and visible debris has been removed.
	Apply a gauze saturated with IQ CARE WOUND SOLUTION and <b>compress</b> for up to 5 minutes every time the dressing is changed.
	Select an appropriate wound dressing as per international guidelines.
	Remember that covered moist wounds heal faster than dry or wet wounds.

<b>SIDE EFFECTS</b>	None known.
	Contains no harmful chemicals or alcohol.
	For external use only.



For any more information visit our website at [www.iq-greensolutions.com](http://www.iq-greensolutions.com)



## **CONTACT DETAILS**

**IQ Green Solutions (Pty) Ltd**

Unit 12, Urban Growth Park

15 Jan van Riebeeck Drive

Paarl, Western Cape

South Africa

Phone: +27(0)21 3000 530

[www.iq-greensolutions.com](http://www.iq-greensolutions.com)

[info@iq-greensolutions.com](mailto:info@iq-greensolutions.com)



**SCAN HERE FOR  
OUR ONLINE STORE**